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Unit:	O mmol/L) mg/dL
Graph range:	54.0	- 216.0
Manually scale	🗹 Time	Glucose
Target range:	70.200005	- 180.0
Glucose-axis numbers LEFT		Scanning starts app
Send to Libreview	Send to xDrip	NFC sound
✓ Invert screen		Reminders
Sensor via bluetooth		Alarms
Number Labels	Colors	

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Dark mode []							
The right of the middle menu contains display options. To view the last scan, touch the first item. A cross after <i>Scans</i> means that scans are displayed in the graph. A scan is the current glucose value you get when you scan your sensor. Scanning also transfers 15 minutes measurements of the last 8 hours from sensor to smartphone. Display of these values can be controlled by touching <i>History. Stream</i> stands for glucose values send every minute via Bluetooth from a Freestyle libre 2 sensor to this applet.							
You can enter all kind of numbers (with whatever label you like) to document you insulin, carbohydrate and activities. They are displayed at fixed height in the glucose graph at the specified time and date. Touching on points in the graph or an entered number gives information about that item. You can edit an entered number by long touching it. Editing is also possible via the number list in the left middle menu and touching the corresponding data element. <i>Amounts</i> determines the display of entered numbers. In Settings, you can specify which labels you like to use for these numbers. Above one of the numbers the corresponding label is shown.							
Menu 2: Left of middle							
Notify []							
οκ							

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Port	7117		IPs:		Detect
127.0.0.1					
🗹 Test IP	ר 🔽	Test Label		libre3new	
O Passive only		O Active only			🔘 Both
Receive from	Send to	Amounts	۵] Scans	Stream
Password		······			Uisible
Delete	Cancel		Help		Save

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